

Opalescence
● ● ● go[®]

POWERFUL,
PROFESSIONAL
WHITENING.

TO GO!



Opalescence Go whitening features ready-to-go convenience with the UltraFit[™] tray for an even more adaptable and comfortable whitening experience.

- Unique tray material offers a remarkably comfortable fit and easily conforms to any patient's smile
- Molar-to-molar coverage ensures the gel comes in contact with more posterior teeth
- UltraFit tray's superior adaptation ensures the maximum amount of gel stays in contact with the teeth during whitening
- Available in 10% hydrogen peroxide (with wear times from 30–60 minutes) and 15% hydrogen peroxide (with wear times from 15–20 minutes)



ULTRADENT
PRODUCTS, INC.

800.552.5512 ULTRADENT.COM



Additional features of Opalescence Go Whitening

- Delicious Mint and Melon flavors
- Convenient prefilled trays can be worn right out of the package
- Optimal gel quantity allows easy cleanup after whitening

6pk Patient Kits (10 treatments per kit)

- 4635 - 10% Mint 6pk
- 4636 - 10% Melon 6pk
- 4638 - 15% Mint 6pk

12pk Mini Kits (4 treatments per kit)

- 4645 - 10% Mint 12pk
- 4646 - 10% Melon 12pk
- 4648 - 15% Mint 12pk

Sample Towers

- 4652 - 10% Mint/Melon Sample Tower
- 4654 - 10% Mint Sample Tower

Refrigeration recommended

Instructions



1. Remove from package.



2. Center tray on arch.



3. Gently suck down or swallow.



4. Remove outer tray.



5. Suck down or swallow again.



6. Wear 6% for 60 to 90 minutes, 10% for 30 to 60 minutes, and 15% for 15 to 20 minutes.

