
EN

Endo-Eze® tips

Gauges: 18, 19, 20, 22


Use with/for: Delivery placement of sealers, luting materials, irrigants, or vacuuming and/or air/water delivery; Luer Vacuum Adapter, TriAway™ adapter and Ultradent® syringes.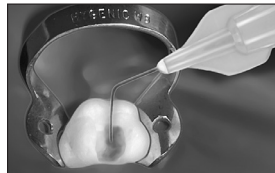

Endo-Eze® 31 gauge tips

Smallest irrigating tip available.


Use with/for: Irrigation with ChlorCid® and application of File-Eze®.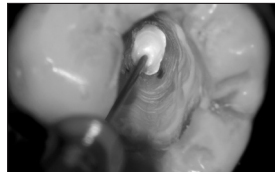

NaviTip®

Strong/rigid 30ga cannula with soft, flexible rounded tip for navigation of curved canals.

Use with/for: File-Eze, ChlorCid, UltraCal® XS, EndoREZ™, EndoSeal™, root canal irrigation/lubrication.

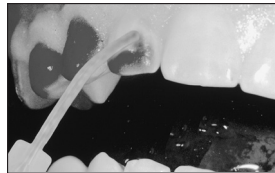SST™ (Surgical Suction Tip)

Large diameter evacuation tip for delicate surgeries.

Use with Luer Vacuum Adapter for small periodontic or endodontic surgery.

Black Mini® tips

Use with/for: TriAway, small restorations in difficult to reach areas, air and/or water.

Skini™ syringe

Use with/for: Luting resins to post canals, crowns, etc.

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
U.S. federal law restricts this device to sale by or on the order of a dentist.

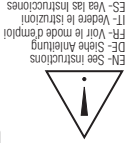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EN - Precautions: See instructions that are included with listed products for use and precautions.

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Micro Capillary™ Tip

Flexible, 0.008" diameter, depth limiting (10mm) plastic tip.

Use with/for: Endodontic evacuation and irrigation, Ultracare®, periodontal material delivery.

5ml/1.2ml syringes

For delivery of root canal irrigants, lubricants, and other dental solutions.

Use with/for: Ultracare, ViscoStat®, ViscoStat® Plus, Astringedent®, Astringedent® X, Buffered Aluminum Chloride; Endo-Eze tips, Capillary tips, etc.

TriAway™

See included instructions.

For tiny preparations, bonding, and luting posts.

Luer Vacuum Adapter

Insert in high volume evacuation hose. Attach desired tip for evacuation of endodontic or surgical sites. Luer Vacuum Adapter insert may be removed and inserted in saliva ejector hose for use as endo/surgical evacuator. Luer Vacuum Adapter is autoclavable.

Capillary™ Tip

Flexible 0.014" tip for evacuation. Use with Luer Vacuum Adapter for evacuation and drying of root canals.

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