PermaFlo® Pink

EN-Flowable Composite
DE-Flüssiges Komposit
FR-Ciment dentaire composite liquide

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PermaFlo® Pink Flowable Composite

Description & Indications:

PermaFlo Pink is a light-cured, radiopaque, methacrylate-based flowable composite. PermaFlo Pink contains 1um average particle size with narrow upper limit particle distribution. It is thixotropic, 68% filled.

Indications:

Use PermaFlo Pink to mask gingival recession where root structure is exposed.

Procedure:

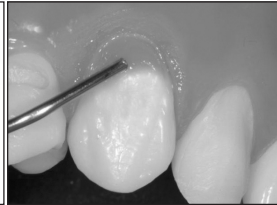
1. Remove luer lock cap and attach a disposable Micro 20 gauge tip securely onto syringe.
2. Isolate preparation with cord and or rubber dam to control sulcular fluid.
3. Etch 20 seconds.
4. Apply dentin bonding agent (We recommend PQ1). Air thin, light cure 20 seconds.
5. Apply first layer of PermaFlo Pink 0.5 - 1 mm and light cure. Follow with 2 mm increments as needed. Light cure each resin layer 20 seconds, see Fig.1.
6. Shape with appropriate finishing burs. Finish with strips, polishing cups, and wheels or disks as appropriate. We recommend Jiffy® Polishing system for shine, ending with Jiffy HiShine, see Fig. 2.

Precautions:

1. Partially cured material will have deficient physical properties, including strength, wear resistance retention, marginal integrity and color stability.
2. Incompletely cured material could also cause pulpal irritation. Use only high-quality curing units check units periodically with a light meter.

3. The oxygen inhibition layer should not be removed or disturbed during incremental layering because it optimizes the bond to each succeeding layer.

4. Any resin can be sensitizing to skin or can cause allergic reaction if individual is already sensitized. Therefore, wash area of skin exposed to resin completely to remove material. If dermatitis, rash or other allergic reaction is observed, see a physician.
5. Syringe caps should be replaced following use to prevent polymerization (as composite resins are activated).

EN Isolate preparation with retraction cord and/or rubber dam. Etch 20 seconds.

Apply puddled coat of PQ1, air thin, light cure 20 seconds.

Adapt PermaFlo Pink to appropriate contour. Verify no overhanging margins are present.

Complete contouring with appropriate burs, cups, disks, etc. Finish with Jiffy HiShine.

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FI Isolate preparation with retraction cord and/or rubber dam. Etch 20 seconds.

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Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512. Outside U.S. call (801) 572-4200 or visit www.ultradent.com.

For product SDS
 Please visit our website:
www.ultradent.com

EN- Recharge
 DE- Nachfüllen
 FR- Recharger
 NL- Aanvullen
 IT- Ricaricare
 ES- Recargar
 PT- Recarregar
 SV- Fylla på
 DA- Genfyld
 FI- Täydentää
 EL- Αναπληρώστε το

0 = 10 min
 1 = 2 min
 2 = 3 min
 3 = 4 min
 4 = 5 min
 5 = 10 min

HAZARD ALERTS

Reactivity

Flammability

Health

General Precautions:

- For Professional use only.
- Review instructions, precautions, and SDS before beginning treatment. Use only as directed.
- Keep products out of heat/sunlight.
- Avoid skin exposure to resins.
- Isolate strong chemicals to area of treatment.
- Confirm that patient has no known allergies to treatment materials.
- Test flow of materials from syringe and tip before using intraorally.
- Never force syringe plungers.
- Use Ultradent® Syringe Covers and/or clean and disinfect syringes between patients.
- Use standard curing lights with all light polymerized/activated materials.

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