Mosaic™ Universal Composite

Descripción: **Mosaic** is a light-cured, bis-GMA-based universal restorative composite. **Mosaic** is radiopaque and available in multiple shades. It has a nano-hybrid composite containing zirconia-silica glass ceramic and 20 nanometer silica. Filler load is 68% by volume for the dentin shades and 56% for the enamel shades. **Mosaic** is supplied in multi-use syringes (4.0 grams) and single compules (0.2 grams).

Indications for use:

Mosaic is used for direct and indirect restorations (inlays, onlays and veneers) in both the anterior and posterior regions.

Directions for Use:

1. Refrigerated, bring product to room temperature.
2. Apply dentin bonding agent to preparation according to Manufacturer's instructions.
3. OPTIONAL: Place a super adhesive layer of flowable composite, like PermaFlo before placing **Mosaic**.

For Syringes:

- A. Flip open cap on the **Mosaic** syringe.
 - B. Twist handle clockwise (stem) to express composite. Resistance will lessen with each twist.
 - C. Immediately replace cap after dispensing composite to prevent polymerization from ambient light.
- NOTE:** Syringes are not for intra-oral use. To avoid cross-contamination use clean gloves and a clean instrument.

For Singles (compules):

- A. Keep tip caps on until use.
- B. Place syringe into dispensing gun.
- C. Make sure tip is firmly snapped into composite gun before trying to express.
- D. Unit dose singles are designed for single patient use only. Do not reuse to avoid the risk of cross contamination. Discard compule after use.
- E. Place **Mosaic** in incremental layers applying the air layer increment (0.5 – 1.5mm thick and a cure. Follow with up to 2mm increments curing after each increment. If curing with VALD or another high quality curing light (>600mW/cm²), cure each layer for 10 seconds. For lights with an output <600mW/cm², cure for up to 20 seconds. A final cure of 20 seconds is suggested with high quality curing lights.
- F. After final curing, remove, sculpt and light cure.
- G. Shape and define the final anatomy using finishing diamonds or multi-fluted carbide burs. Finish and polish with polishing discs, cups, points or brushes.

A composite sealer can be placed over restoration according to Manufacturer's instructions to enhance sheen and reduce surface wear.

Precautions and Warnings:

1. Read and understand all instructions before using product.
2. For daily use, store at room temperature. Refrigerate bulk storage.
3. Use a quality curing light (if curing with VALD or another high quality curing light (>600mW/cm²), cure each layer 10 seconds. For lights with an output <600mW/cm², cure for 20 seconds.) to ensure maximum physical properties of the composite. Uncured material will result in an inferior restoration and may also cause pulp irritation.
4. Clinician and patient should wear UV protective eyewear when using composite.
5. To lessen pre-polymerization of the composite, minimize exposure to bright lights during the procedure.
6. To optimize bond to each succeeding layer, do not remove oxygen inhibition layer between incremental layers of composite.
7. Enamel composite is totally polymerized between layers and intra-proximally.
8. Resins can be sensitizing. Avoid repeated contact of uncured dental resin with skin. Do not use on patients with a known sensitivity to acrylates or other resins. If allergic reaction, dermatitis or rash develops, consult a physician.
9. For Syringes are not for intra-oral use. To avoid cross-contamination use clean gloves and a clean instrument.
10. Disinfect contaminated syringes with an intermediate level disinfectant between uses.
11. Property dispute of syringes and compules.
12. Keep away from extreme heat and direct sunlight.
13. Do not use after expiration date noted on containers.

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