Before using, allow product to warm to room temperature. This will make it easier for the tray to adhere to the teeth. Remove from package.

†^U - Upper whitening tray
†^L - Lower whitening tray

Center tray on arch.

Gently suck down on the tray. Do not press tray against teeth as it pushes the whitening gel against the gums, causing tissue irritation.

Remove outer tray.

Suck down or swallow again.

Wear Opalescence Go 6% for 60-90 minutes per day. When finished remove whitening tray and brush teeth.

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Description:

Opalescence Go™ is a dentist-controlled, dentist-supervised, take-home whitening system, dispensed in pre-loaded disposable trays. It is a clear, fluoride, high viscosity, sticky, 6% hydrogen peroxide gel with potassium nitrate and sodium fluoride.

Precautions for Europe:

- Not to be used on a person under 18 years of age.
- To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.
- Contains sodium fluoride.
- Please note the expiration date printed on the outer package/tray containers.

Precautions:

- If you have any questions regarding the appropriate use of this product, including how long it will take to whiten your teeth, please consult your dentist.
- Regular dental checkups and cleanings are important before and after whitening to maintain a healthy smile.
- Some patients experience sensitivity with whitening. Should this occur, stop treatment and contact your dentist.
- Some patients have noticed temporary discomfort of the gums, lips, throat, or tongue. Should any of these symptoms persist more than two days or progressively worsen, call your dentist. These side effects will usually subside within 1-3 days after treatment is discontinued.
- Pregnant or breastfeeding women, or patients with serious health concerns should not whiten. Consult their primary care provider prior to treatment
- If patient has a known allergy or chemical sensitivity to any of the ingredients in Opalescence whitening, do not use. Mint or melon flavor can be irritating to some patients. If a patient has a known sensitivity to flavorings, the non-flavored variant should be selected.
- Do not use household bleach to whiten teeth.
- Do not use tobacco or eat while whitening trays are in place.
- Teeth are naturally darker along the gum line. These areas may require more time to lighten than the rest of the tooth surface and usually remain slightly darker.
- White spots that appear on the enamel during the whitening process may blend with continued whitening.
- Crowns, bridges, partial dentures, veneers, and composite fillings will not whiten.
- Coffee, tobacco, and other products can re-stain your teeth over time. Should this occur, teeth can be re-whitened.
- Some amalgam fillings may leave a dark purple color in your whitening tray, this is normal.
- Check expiration date and store according to instructions.
- Avoid swallowing large amounts of gel or water used to rinse gel residue from the teeth.
- Avoid contact with eyes.
- Do not reuse to avoid cross contamination.

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