
InterGuard®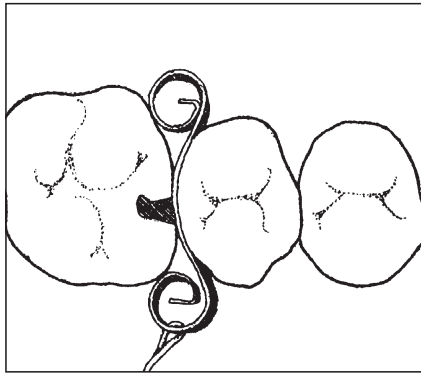

English
To prevent aspiration, use rubber dam or dental floss (18cm).

Place the InterGuard between teeth. Adjust the curls for a firm fit. With tight proximal contact, separate with a wedge or a retaining ring.

Place InterGuard with curls turned to tooth being prepared.

At the end of the preparation InterGuard will loosen. If necessary stabilize with a finger or stabilize during the whole procedure with a retaining ring (in this case turn the curls away from the preparation). Do not reuse to avoid cross contamination.

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InterGuard®

Description & Indications:

InterGuard® is a preventive aid for operative dentistry. It is used to protect adjacent teeth from iatrogenic preparation damage. InterGuard is designed for autoclave and dry heat sterilization. Made of stainless steel with a curl at each end, InterGuard fits snugly in both large and small interproximal spaces. InterGuard is available in two heights:

- 4.0mm for short anatomical crowns.
- 5.5mm for tall anatomical crowns.

Procedure:

1. To prevent patient from accidentally swallowing InterGuard use a rubber dam or fasten a piece of dental floss through the hole on the side of InterGuard. To aid in retrieval make sure that the floss hangs no less than 7-inches outside of mouth.
2. Adjust the space between the curls with your fingers to allow InterGuard to fit firmly between the teeth.
3. Place InterGuard with curls toward the tooth to be prepared (Fig. 1). The curls keep the InterGuard clear of the transition angles, allowing full access to the area to be prepared.
4. With tight proximal contacts, use wedges

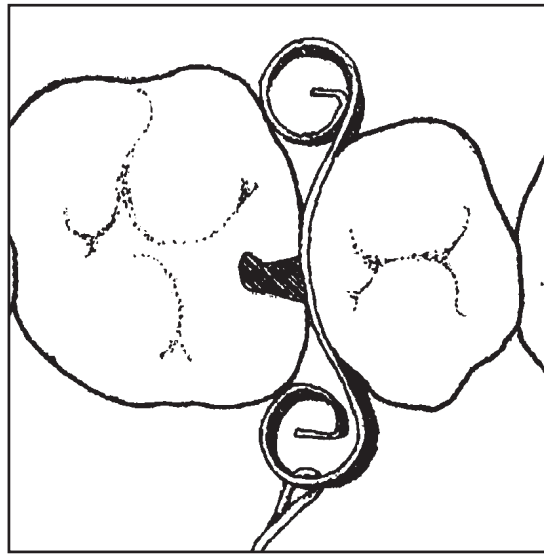
Fig. 1 Place InterGuard curls toward the tooth to be prepared.

to simplify placement or separate with a retaining ring.

Note: At the end of preparation, InterGuard will loosen. If necessary, stabilize with a finger or InterGuard may be stabilized during the entire procedure with a retaining ring (in this case turn curls away from the tooth being prepared).

Precautions:

1. Always isolate the preparation area with a rubber dam or ligate InterGuard with a 7-inch length of dental floss to prevent aspiration.
2. For professional use only.

EN- Use by date
DE- Losnummer
FR- A utiliser avant le
NL- Uiterste gebruiksdatum
IT- Data di scadenza
ES- Fecha de caducidad
PT- Data de validade
SV- Förbrukningsdatum
DA- Anvendes inden
FI- Vuimeinen käyttöajankohta
EL- Ανάλωση ζωας

EN- Catalog Number
DE- Katalognummer
FR- Numéro de catalogue
NL- Catalogusnummer
IT- Numero catalogo
ES- Número de Catálogo
PT- Número de catálogo
SV- Katalognummer
DA- Katalognummer
FI- Tuotenumero
EL- Αριθμός καταλόγου

EN- For Professional use only.
DE- Nur für professionelle Anwendungen.
FR- Destiné à une utilisation professionnelle uniquement.
NL- Uitsluitend voor professioneel gebruik.
IT- Solo per uso professionale.
ES- Sólo para uso profesional.

Do not re-use to avoid cross contamination
DE- Nicht wiederholt verwenden, um Kreuzkontamination zu verhindern.
FR- Ne pas réutiliser pour éviter la contamination croisée.
NL- Niet opnieuw gebruiken om kruisbesmetting te vermijden.
IT- Per evitare la contaminazione crociata non riutilizzare.
ES- No lo reutilice, así evitará la contaminación cruzada.
PT- Não reutilizar para evitar a contaminação cruzada.
SV- För undvikande av korskontamination får produkten inte återanvändas.
DA- Må ikke genbruges for at undgå krydskontaminering.
FI- Älä käytä uudelleen riskikontaminaation välttämiseksi.
EL- Μην το επαναχρησιμοποιείτε για να αποφύγετε τη διασταυρωτική μόλυνση.

Key:

EN- See instructions
DE- Siehe Anleitung
FR- Voir le mode d'emploi
NL- Zie gebruiksaanwijzingen
IT- Vedere le istruzioni
ES- Vea las Instrucciones
PT- Ver instruções
SV- Se bruksanvisning
DA- Se vejledningen
FI- Lue käyttöohjeet
EL- Δείτε τις οδηγίες

EN- Lot Number
DE- Produktionsnummer
FR- Numéro de lot
NL- Partijnummer
IT- Numero lotto
ES- Número de lote
PT- Número de lote
SV- Sats nummer
DA- Partinummer
FI- Eränumero
EL- Αριθμός παρτίδας

Ultradent Products GmbH, Am Westhoyer
Berg 30, 51149 Cologne Germany.

EN- Recycle
DE- Recyceln
FR- Recycler
NL- Recyclen
IT- Riciclare
ES- Reciclar
PT- Reciclar
SV- Recycla
DA- Genbringe
FI- Kierrättää
EL- Ανακυκλώστε το

U.S. Federal law restricts the sale of this device by or on the order of a state licensed dental professional.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free Outside U.S. call (801) 572-4200.

InterGuard®

Manufactured by:
Ultradent Products Inc
Made in USA

800.552.5512 ULTRADENT.COM

ULTRADENT
PRODUCTS, INC.

Copyright 2011 Ultradent Products, Inc./InterGuard ApS. All Rights Reserved.