EN - Keep out of reach of children
 DE - Außerhalb der Reichweite von Kindern aufbewahren
 FR - Tenir hors de portée des enfants
 NL - Buiten bereik van kinderen houden
 IT - Tenere fuori dalla portata dei bambini
 ES - Manténgase fuera del alcance de los niños
 PT - Manter fora do alcance das crianças
 SV - Förvaras utom räckhåll för barn
 DA - Opbevares utilgængelig for børn
 FI - Pidettävä pois lasten ulottuvilta
 EL - Φυλάξτε μακριά τα παιδιά

EN - Toxic
 DE - Giftig
 FR - Toxique
 NL - Giftig
 IT - Tossico
 ES - Tóxico
 PT - Tóxico
 SV - Giftigt
 DA - Toksik
 FI - Myrkyllinen
 EL - Τοξικό

EN - Irritant
 DE - Reizmittel
 FR - Irritant
 NL - Irriterend
 IT - Irritante
 ES - Irritante
 PT - Irritante
 SV - Irriterande
 DA - Irritant
 FI - Ärsyttävä aine
 EL - Ερεθιστικό

Imported By:
 Clinician's Choice Dental Products / Clinical Research Dental
 NBA 1M6
 Syringe NPN 80011364
 Tray NPN 80031866

EN Do not reuse to avoid cross contamination.
 DE Nicht wiederholt verwenden, um Kreuzkontamination zu verhindern.
 FR Ne pas réutiliser pour éviter la contamination croisée.
 NL Niet opnieuw gebruiken om kruisbesmetting te vermijden.
 IT Per evitare la contaminazione crociata non riutilizzare.
 ES No lo reutilice, así evitará la contaminación cruzada.
 PT Não reutilizar para evitar a contaminação cruzada.
 SV För undvikande av korskontamination får produkten inte återvändas.
 DA Må ikke genbruges for at undgå krydskontaminering.
 FI Älä käyttää uudelleen ristikontaminaation välttämiseksi.

EL Μην τοποαναχρησιμοποιείτε για να αποφύγετε τη διασταυρωτική μόλυνση.
 EN For Professional use only.
 DE Nur für professionelle Anwendungen.
 FR Destinée à une utilisation professionnelle uniquement.
 NL Uitsluitend voor professioneel gebruik.
 IT Solo per uso professionale.
 ES Solo para uso profesional.
 PT Destina-se apenas à utilização profissional.
 SE Endast för yrkesmässigt bruk.
 DA Må kun anvendes af tandlægepersonale.
 FI Vain ammattikäyttöön.
 EL Μόνο για επαγγελματική χρήση.

UltraEZ[®] Flor-Opal[®]

EN- Tooth Desensitizing agents
 DE - Zahndesensibilisierungsmittel
 FR - Agents de désensibilisation des dents
 NL - Tanddesensitiseringsmiddelen
 IT - Agenti di desensibilizzazione dentaria
 ES - Agentes desensibilizadores para los dientes
 PT - Agentes de dessensibilização dentária
 SV - Medel för desensibilisering av tänder
 DA - Desensibiliserende midler til tænder
 FI - Hampaiden arkuutta vähentävät aineosat
 EL - Ουσίες απευαισθητοποίησης των δοντιών

800.552.5512 ULTRADENT.COM

ULTRADENT
 PRODUCTS, INC.

© Copyright 2011 Ultradent Products, Inc. All Rights Reserved.

Manufactured by Ultradent Products, Inc. Made in USA

1005 West 10200 South, South Jordan, Utah 84095 41300.17 082012

EN DE FR NL IT ES PT SV DA FI EL

UltraEZ

EN-Description:

UltraEZ[®] is a unique, sustained release, 3% potassium nitrate and 0.11% weight by weight fluoride ion, sticky, viscous gel. UltraEZ is designed to be used in a custom-fabricated tray.

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Lightly press into place.

Remove outer tray.

After placing tray in mouth, gently press the tray from biting surface of teeth towards the gums to ensure complete coverage of gel.

UltraEZ (Green Tray): Wear for 15 minutes to 1 hour per day or as directed by your dentist.

Flor-Opal (Blue Tray): Wear for 30 minutes to 1 1/2 hours per day or as directed by your dentist.

When finished remove tray and brush teeth.

Precautions:

1. Keep desensitization product away from small children.
2. This product is NOT for nighttime use.
3. Do not use tobacco products or eat while using this product.
4. Not intended for use by pregnant women.
5. If any problems occur, discontinue treatment and notify your dentist.
6. Keep out of heat and/or sunlight at all times. UltraEZ and Flor-Opal may be stored at room temperature.
7. If a large volume of the product is accidentally swallowed, get medical help or contact a Poison Control Center right away.
8. Do not reuse to avoid cross contamination.

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