# PropGard™

Mouth Prop / DE - Mundsperrerr / FR - Protège - langue / NL - Bijtblok / IT - Apribocca / ES - Sostén Bucal / PT - Abre - Bocas / SV - Munöppnare / DA - Mundspærerr / FI - Purutuki / EL - Στοματοδιαστολέας

## Description & Indications:


Use PropGard™ to hold mouth open and retract the tongue simultaneously.

## Procedure:

1. Insert right or left tongue guard in mouth piece. Choose appropriate slot for superior/inferior relationship best suited to each patient.
2. Right and left noted on side of tongue guard. "R" or "L" notation should be to posterior in correct position in patients mouth.
3. Using index finger, press in on tongue guard to retract tongue as needed.

Fig. 1 Assemble PropGard as shown.

Fig. 2 Position PropGard in mouth.

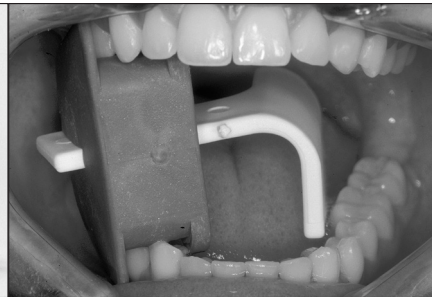Cleaning & Sterilization Processing Instructions		EN		
Initial treatment at the time of use	Sterilization is required before every use.			
Cleaning	Place device in an ultrasonic cleaner with the detergent listed below:			
	Detergent	Cleaning Time		
	Henry Schein General Purpose Cleaner or equivalent product	2-10 mins		
Rinsing	Rinse device with warm water to remove detergent for 1-2 mins.			
Drying	Dry with gauze. Air dry 30 mins.			
Maintenance, inspection and testing	Visually inspect that the device is not bent or damaged and free of debris. If debris is present repeat cleaning process.			
Packaging	Place selected device in approved sterilization packaging (ISO 11407 & ISO 11607).			
Sterilization	Place sterilization package in autoclave using the following settings:			
	Cycle Type	Minimum Sterilization Exposure Time	Minimum Sterilization Exposure Temperature	Minimum Dry Time
	Pre-vacuum	3 mins	132°C /270°F	20 mins
	Do not overload the sterilizer.			
Storage	Keep device in sterilized packaging until ready for use. Keep out of sunlight and moisture.			
Additional information	If sterilization packaging is compromised, repeat sterilization process. This procedure was validated by an independent and accredited laboratory.			

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### PropGard™ / Tongue Guard

Manufactured by:  
Ultradent Products, Inc.  
505 West Ultradent Drive (10200 South)  
South Jordan, Utah 84095 USA  
Made In USA

8 0 0 . 5 5 2 . 5 5 1 2 | ultradent.com  
© 2020 Ultradent Products, Inc. All Rights Reserved. 24617M12 051220

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